

When to Fit Extreme H2O Lenses

Fit Extreme H2O to provide a solution for patients with specific fitting needs who are looking for improved comfort. All products are made with non-ionic, ultra-hydrating materials that retain up to 99% of their moisture, keeping eyes fresh and hydrated all day long.



Extreme H2O Daily

The **ONLY** daily disposable available in **13.6** **14.2** **14.8**. Successfully fit your small, medium, and large cornea patients with the right size lens.

- Patients who would benefit from a smaller or larger diameter
- Patients who have an HVID ≤ 11.3 or HVID ≥ 12.3 (27% of patients fall in this range)
- Patients who are uncomfortable in their current lens



Extreme H2O Weekly

The **ONLY** one-week disposable available in **13.6** **14.2** **14.8**. Successfully fit your small, medium, and large cornea patients with the right size lens.

- Patients who would benefit from a smaller or larger diameter
- Patients who have an HVID ≤ 11.3 or HVID ≥ 12.3 (27% of patients fall in this range)
- Patients who are uncomfortable in their current lens
- Patients who are looking for more affordable frequent replacement

Extreme H2O 54

Available in spherical and toric products with unique options for your patients' individual needs.

- 13.6 - the smaller lens design is a great solution for pediatric eyes, patients with tight lids, and patients who have difficulty inserting and removing their lenses
- Toric LC - designed for the 55% of patients with astigmatism from 0.50D to 1.00D - monthly replacement suggested

Extreme H2O 59

A two-week replacement problem-solving lens for patients who experience extreme dryness and discomfort.

- Middle-aged users and patients living in low humidity areas
- Patients taking certain medications and allergy sufferers
- Patients who work in climate controlled offices and/or computer users

Extreme H2O Monthly

A monthly lens with 3 base curve options for an optimum fit.

- Patients who need a comfort upgrade from Group 4 or SiHy lenses
- Patients who want an affordable wearing option

Extreme H2O Product Specifications

Extreme H2O Daily



13.6 Diameter	Median/8.2 BC	Powers: +0.50 to +6.00 & -0.50 to -10.00
14.2 Diameter	Median/8.5 BC	Powers: +0.50 to +6.00 & -0.50 to -10.00
14.8 Diameter	Median/8.7 BC	Powers: +0.50 to +6.00 & -0.50 to -10.00 0.070 mm @ -3.00 center thickness

Extreme H2O Weekly



13.6 Diameter	Median/8.2 BC	Powers: +0.50 to +6.00 & -0.50 to -10.00
14.2 Diameter	Median/8.5 BC	Powers: +0.50 to +6.00 & -0.50 to -10.00
	Steep/8.3 BC	Powers: -0.50 to -10.00
14.8 Diameter	Median/8.7 BC	Powers: +0.50 to +6.00 & -0.50 to -10.00 0.110 mm @ -3.00 center thickness

Extreme H2O 54

13.6 Diameter	Median/8.3 BC	Powers: +0.50 to +6.00 & -0.50 to -12.00 0.081 mm @ -3.00 center thickness
14.2 Diameter	Median/8.6 BC	Powers: +0.50 to +6.00 & -0.50 to -11.00 0.085 mm @ -3.00 center thickness

Extreme H2O 54 Toric

14.2 Diameter	Median/8.6 BC	Powers: +4.00 to -6.00
LC - Low Cylinder (-0.65):	15° to 180° (in 15° steps)	
MC - Mid Cylinder (-1.25):	10° to 180° (in 10° steps)	
		0.145 mm @ -3.00 center thickness

Extreme H2O 59

14.2 Diameter Thin	Median/8.6 BC	Powers: +0.50 to +6.00 & -0.50 to -8.00 0.085 mm @ -3.00 center thickness
14.2 Diameter Xtra	Median/8.6 BC	Powers: +0.50 to +6.00 & -0.50 to -10.00
	Steep/8.3 BC	Powers: -0.50 to -8.00 0.145 mm @ -3.00 center thickness

Extreme H2O Monthly

14.0 Diameter	Flat BC	Powers: -0.50 to -6.00
	Median BC	Powers: +0.50 to +6.00 & -0.50 to -10.00
	Steep BC	Powers: -0.50 to -10.00
		0.090 mm @ -3.00 center thickness

All products: .50D steps after -6.00



X-CEL
SPECIALTY CONTACTS